



CleanRead™ ViB Visual Inspection Borescope

The CleanRead™ ViB (Visual Inspection Borescope) provides clear, reliable views inside endoscopes and cannulated instruments. It helps technicians verify effective cleaning and detect debris, discoloration, or damage that may remain hidden. Easy to use and integrate into routine workflows, the ViB supports quality assurance, patient safety, and the longevity of valuable instruments.



ONE-PASS Camera Wand

Reaches the distal tip through the suction valve, for a full internal inspection.



High-definition imaging

120° field of view and 400×400 resolution for clear visualization.



Auto Brightness Control

Automatically adjusts illumination for consistent clarity.



Waterproof Wand

Fully sealed and easy to clean, for fast turnaround between inspections.

Distributed by:



13211 Stafford Road, Ste 800
Missouri City TX 77489

www.sterilemate.com
info@sterilemate.com

Before You Begin,

1.

Ensure the scope is fully cleaned "at the sink".

2.

Ensure all connectors & cleaning accessories are removed.

3.

Move the scope to the inspection station.

System Setup



Connect the ViB controller to the all-in-one computer with the USB cable.



Open the ViB software and input login credentials.




On the left-hand menu, open Visual Inspection > New Procedure.

Inspection Procedure



Run an inside channel test with your ATP handheld while connected to Wi-Fi.

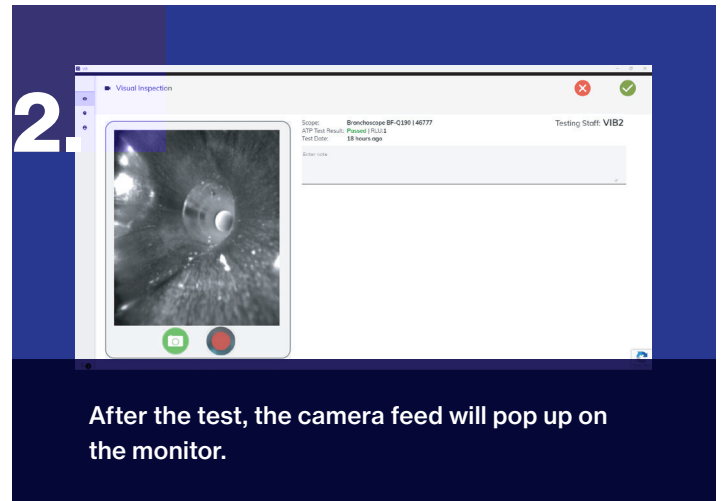

After the test, the camera feed will pop up on the monitor.



Position the distal tip at the channel opening and, using short, controlled strokes, slowly advance it through the channel.



Watch the feed for debris, discoloration, tears, or scratches. Press the GREEN button to capture images.


Distributed by:



13211 Stafford Road, Ste 800
Missouri City TX 77489

www.sterilemte.com
info@sterilemate.com

After Inspection

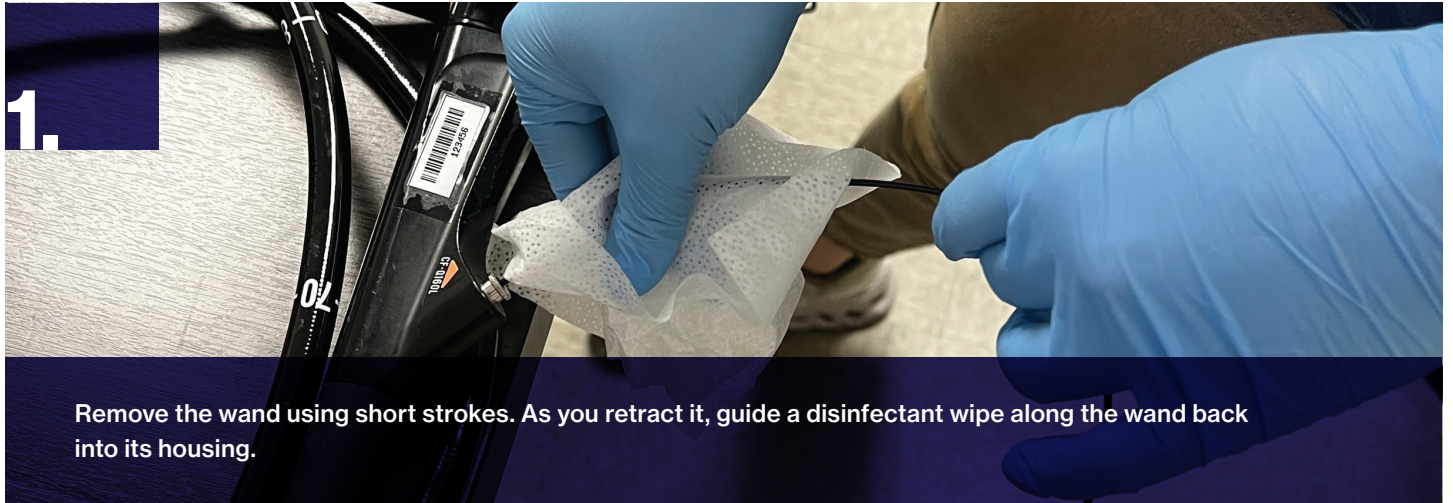

Cleaning Recommendations

Camera Wand

- **70% Isopropyl Alcohol**
Safe for routine surface cleaning and disinfection.
- **Ethylene Oxide (EtO)**
Suitable for sterilization when required by facility protocol.
- **Note:** Avoid submerging the wand; wipe only. Use lint-free cloths to maintain optical clarity.

All-in-One Computer

- **Screen Cleaner (Non-Alcohol, Non-Abrasive)**
Use a microfiber cloth with a dedicated monitor-safe cleaning solution.
- **Note:** Do not apply liquid directly to the screen; spray onto cloth first.

Package Contents



1 x All-in-one computer



1 x Borescope controller



1 x AC adapter for AIO Computer



1 x Power cable for borescope controller



1 x Visual Inspection Borescope



1 x Casing for borescope controller

Distributed by: